


Operation Instruction

VI. Operation Instruction

The press buttons are arranged on the control board for easy of use. The following diagram illustrates the buttons on control board and their functions.

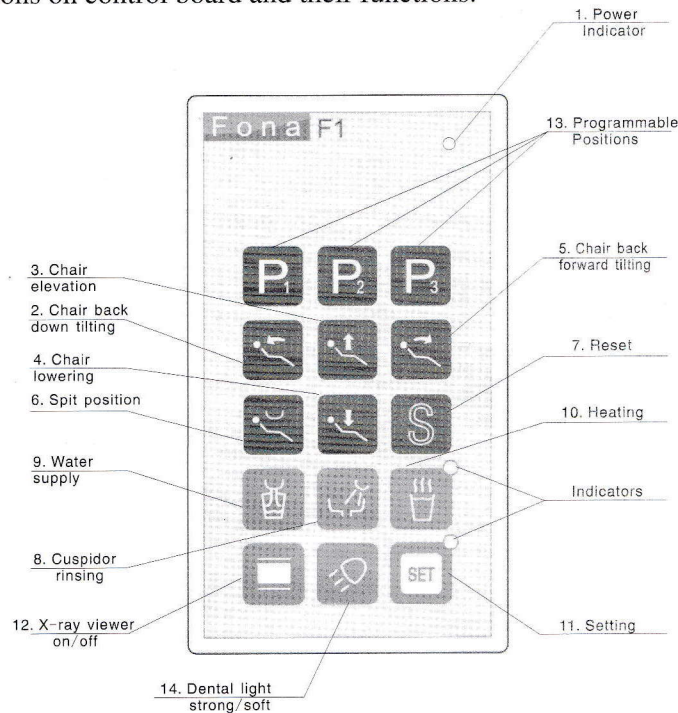

1. Power indicator

Once power is on, the power indicator will illuminate. It will go off when power is off.

2. Chair back down tilting button

By pressing this button and holding on, the back of dental chair will move in the indicated direction. When reaching required position, stop pressing the button and the chair back will immediately stop moving. (If button pressing continues, chair back will automatically stop moving when lower limitation position is reached.)

3. Chair elevation

By pressing this button and holding on, the dental chair will move in the indicated direction. When reaching required position, stop pressing the button and the chair will immediately stop moving. (If button pressing continues, chair will automatically stop moving when upper limitation position is reached.)

4. Chair lowering

By pressing this button and holding on, the dental chair will move in the indicated direction. When reaching required position, stop pressing the button and the chair will immediately stop moving. (If button pressing continues, chair will automatically stop moving when lower limitation position is reached.)

5. Chair back forward tilting button

By pressing this button and holding on, the back of dental chair will move in the direction as indicated. When reaching required position, stop pressing the button and the chair back will immediately stop moving. (If button pressing continues, chair back will automatically stop moving when upper limitation position is reached.)