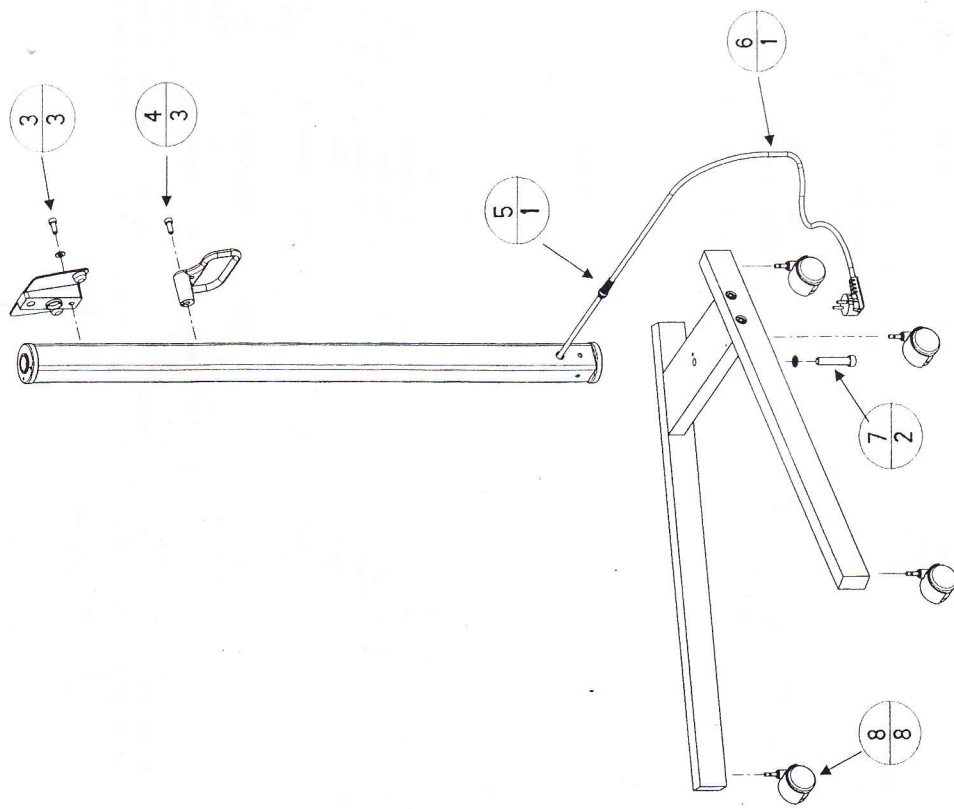
**PESO TOTALE** 32,60 Kg  
**DIMENSIONI IMBALLO BASE STATIVO** 83x24x7 cm  
**DIMENSIONI IMBALLO MONTANTE STATIVO** 116x16x10 cm  
**TOTAL WEIGHT**  
**DIMENSIONS BASE MOBILE PACKAGE**  
**DIMENSIONS VERTICAL SUPPORT MOBILE PACKAGE**

RIF	RICAMBI STATIVO MOBILE SPARE PARTS MOBILE	CODICE ARTICOLO CODE	VERSIONE RAL RAL 9010 VERSION	N° DEI PEZZI Q.TY PCS
A	STATIVO (A) MONTANTE STATIVO	29700394	29700394 C	1
B	STATIVO (B) BASE	29700393	29700393 C	1
1	STATIVO (1) SET VITI	29701049	29701050	21
2	STATIVO (2) SET ACCESSORI	29701050	29701098	4
3	STATIVO (3) SUPPORTO SUPERIORE	27501076	27501099	3
4	STATIVO (4) SUPPORTO INFERIORE	27501077	33800222	2
5	STATIVO (5) BLOCCACAVO	33800222	32000254	1
6	STATIVO (6) CAVO ALIMENTAZIONE	32000254	29701051	1
7	STATIVO (7) VITE ASSEMBLAGGIO	29701051	29700446	2
8	STATIVO (8) RUOTA STANDARD (4 pz)	29700446	29700447	8
8	STATIVO (8) RUOTA STANDARD	29700447	29700442	2
8	STATIVO (8) RUOTA GRANDE ANT. CON FRENO	29700442	29700443	1
8	STATIVO (8) RUOTA GRANDE POST	29700443	29700443	1

Via Roma 45  
 21057 OLGiate OLGINA VA ITALY

stativo mobile  
 mobile unit

**de Götzen® S.r.l.**  
 Via Roma 45  
 21057 OLGiate OLGINA VA ITALY  
 Tel. +39 0331 376760 r.a.  
 Fax +39 0331 376763

E-mail: [degotzen@degotzen.com](mailto:degotzen@degotzen.com)

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The MOBILE is a steel structure painted with epoxy dust, shock proof and anti-abrasion, consisting of a BASE and a VERTICAL SUPPORT.

MOBILE moving is allowed by an handle, fitted to the VERTICAL SUPPORT, and by wheels, fitted to the BASE.

The shape and weight of the MOBILE prevent its upsetting.

### ATTENZIONE

About the informations not explained in this document and related to safety advisements, warranty conditions, manufacturer's responsibility, installation, use and maintenance, consult the manuals attached to the "de Gotzen" 's radiographic system, enclosed with this application.

1. open the VERTICAL SUPPORT packaging (116x16x10cm)
2. draw the timer upper and lower holder and the two plastic bags where screws (SCREWS SET) and accessories (ACCESSORIES SET) required for the installation are contained

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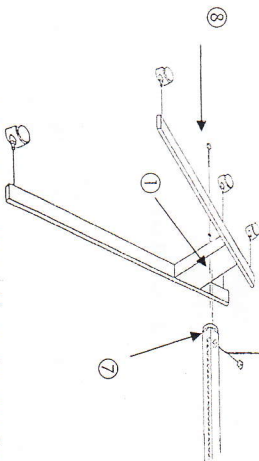
The ACCESSORIES SET contains:

- assembly instructions
- n° 4 standard wheels

3. open the BASE packaging (83x24x7cm)
4. arrange, on a flat surface, the vertical support ① with the blind hole towards the bottom

5. set laterally the supports ②③ paying attention that the wheel inserting caps ⑥ are towards the top and the lateral holes for the screws are aligned to the lateral holes of the central support ①
6. assemble the central support ① with the lateral supports ②③: insert the screws ⑤ with the relevant washers ④ using the provided key, tight alternately
7. turn the BASE, laying it on the side
8. take the VERTICAL SUPPORT
9. locate, on the VERTICAL SUPPORT, the side where the elastic pin ⑦ is mounted
10. insert the elastic pin ⑦ in the blind hole of the central support ①

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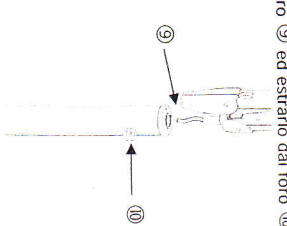
11. on the opposite side of the central support, insert the screw ⑧ and its relevant washer into the hole using the provided key, tight firmly
12. insert the wheel (\*) pins into the suitable caps

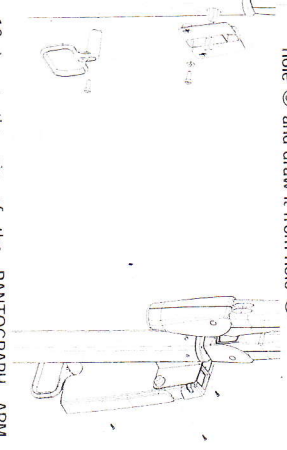
### PLEASE NOTE

(\*) Wheels are supplied in the standard size

BIG WHEELS

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13. lift the MOBILE from the assembling plane and lay it with the wheels placed on the round
14. take the PANTOGRAPH ARM
15. insert the cable coming from PANTOGRAPH ARM into the hole ⑨ and draw it from hole ⑩

16. insert the pin of the PANTOGRAPH ARM into the VERTICAL SUPPORT
17. fix the timer upper holder and the timer lower holder to the VERTICAL SUPPORT using the provided screw and key, tight firmly
18. take the timer and open the timer
19. bring the bottom part of the timer near to the holders
20. insert all the cables coming from the hole ⑩

### PLEASE NOTE

The power supply cable is already inserted into the VERTICAL SUPPORT

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